

# HAMA-JET™ WEEKLY

مطهر و منظم أجهزة الشفط - أسبوعي

## Weekly Aspirator Cleaner and Disinfectant



**For the Weekly cleaning and maintenance of Dental Aspirator Systems.  
Can be used on wet line, dry line and semi wet/dry line suction systems.**

### Key Product Features

Highly effective cleaning and disinfecting product. Also used for effective cleaning of the spittoon bowl. Non-foaming and non-abrasive aldehyde, chlorine and phenol free solution. Easy to use and measure. Colour coded. Acid based to remove protein and organic residues.

### Directions for use

Dispense 5% solution: 50 ml weekly aspirator cleaner concentrate to 950ml warm water. Aspirate via the high and low volume suction tubes and also dispense into the spittoon bowl. Leave for a minimum of 10 minutes in order to activate the working properties and then rinse thoroughly with clean water.

Time of action 10-60minutes

### Composition

Weekly Aspirator concentrate contains Citric Acid, C12-C14 Alkyldimethylammonium chloride.

### Physical Properties

Green liquid, odour characteristic, pH 9-10, soluble in water, Shelf life of 3 Years.

### Packaging

1111-073 1 Litre Container bottle  
1111-022 5 Litre Container  
Constrate 50 ml Equals 1 Litre  
Constrate 1 Litre Equals 20 Litre  
Constrate 5 Litre Equals 100 Litre

### Effective Spectrum

✓	Bacteria
●	MRSA
●	E-Coli
●	Pseudomonas aeruginosa
●	Enterococcus hirae
●	Staphylococcus Aureus
✓	Fungus
●	Aspergillus niger
●	Candida albicans

